INSTRUCTIONS FOR USE | INSTRUCTIONS D'UTILISATION | ISTRUZIONI
PER L'USO | ANWENDUNGSHINWEISE | INSTRUCCIONES DE USO |
INSTRUÇÕES DE UTILIZAÇÃO | INSTRUCIONES DE USO | ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ |
BRUGSVEJLEDNING | GEBRUIKSAANWIJZING | BRUKSANVISNING |
KÄYTTÖOHJEET | BRUKSANVISNING | INSTRUKCJE UŻYTKOWANIA | NÁVOD
K POUŽÍTÍ | HASZNÁLATI UTASÍTÁS | NÁVOD NA POUŽITIE | NAVODILA
ZA UPORABO | LIETOŠANAS INSTRUKCIJA | NAUDOJIMO INSTRUKCIJOS |
KASUTUSJUHEND | INSTRUCIUNI DE UTILIZARE | ИНСТРУКЦИИ ЗА РАБОТА |
UPUTE ZA UPORABU | 使用说明 | ИНСТРУКЦИИ ПО ЭКСПЛУАТАЦИИ |
UPUTSTVA ZA UPOTREBU | KULLANIM TALIMATLARI | 사용 안내

Manufactured for:

Respiroics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668
U.S.A.

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INSTRUCTIONS FOR USE

Intended Use


The EASYLIFE Nasal Mask is an accessory to a non-continuous ventilator (respirator), intended for use by patients >66 lbs (30 kg) prescribed continuous positive airway pressure (CPAP) or bi-level therapy. The mask is intended for single patient use in the home environment or multi-patient reuse in the hospital/institutional environment.

Important: Exhalation vents are built into this mask so a separate exhalation device is not required.

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⚠ Warnings

- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. **Do not block or try to seal the exhalation port.**
- Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed.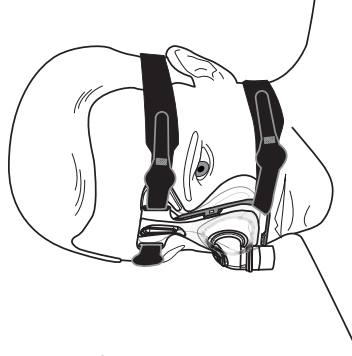

- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. **Explanation of the Warning:** When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, nasal mask selection, and the leak rate. This warning applies to most types of CPAP machines.
- Some users may experience skin redness, irritation and/or discomfort. If this happens, discontinue use and contact your health care professional.
- This mask is not suitable for providing life support ventilation.

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For **single patient reuse** in the home, the mask and CPAP tubing should be cleaned daily. The headgear should be cleaned weekly, or as needed. The headgear does not need to be removed for daily cleaning. Inspect the headgear for wear and tear; replace as needed.

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Achieving the Right Fit

1. Have the patient hold the mask with the inner cushion over their nose and pull the headgear over their head. The headgear is preset to a large setting to go on easily right out of the package.
2. Have the patient evenly adjust the upper straps (Figure 3) to a comfortable position by using the tabs on both sides of the headgear. Do not pull the straps tight. The cushion should lightly touch the face and the upper strap will rest above the ears.
3. Evenly adjust the lower straps (Figure 4) to a comfortable position by using the tabs on both sides of the headgear. Do not pull the straps tight. The lower strap will sit below the ears and the headgear will cross at the base of the head (Figure 5).
4. Connect the tubing (included with the CPAP or bi-level device) to the tubing swivel.
5. Turn on the airflow.
6. Have the patient lie down and breathe normally through the nose, keeping lips closed.
7. Adjust the headgear evenly to reduce leaks or pressure points at the bridge of the nose or above the lip.
8. Reseat the mask, if necessary.

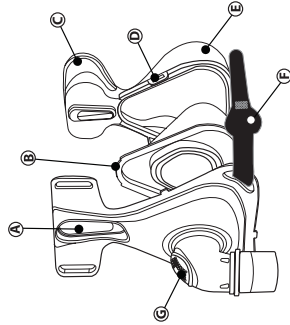
Figure 2

- A = Forehead pad slot
- B = Inner sealing cushion with blue flap
- C = Forehead pad
- D = Attachment tabs
- E = Outer support cushion
- F = Headgear stop
- G = Exhalation (Do not block or remove)

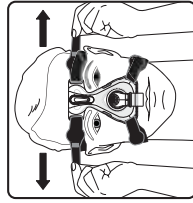
Figure 3

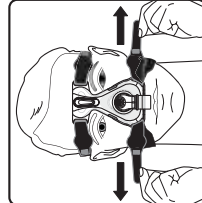
Figure 4

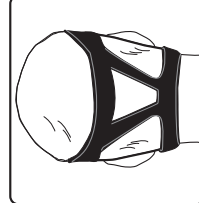
Figure 5

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1054032
JAN 10/23/09

Disposal

Dispose of in accordance with local regulations.

Specifications

Deadspace Volume

S	58 ml
M	65 ml
MW	81 ml
L	83 ml

Storage Conditions

-4° to +140° F (-20° to +60° C)
15% to 95% RH, non-condensing

EASYLIFE SE

English

INSTRUCTIONS FOR USE


Intended Use

The EASYLIFE SE Nasal Mask is intended to provide an interface for application of CPAP or Bi-level therapy to patients. The mask is for single patient use in the home and multi-patient reuse in the hospital/institutional environment. It is intended for use on patients (>66lbs/30kg) for whom CPAP or bi-level therapy has been prescribed.

This mask is latex free and DEHP [Di(2-ethylhexyl)phthalate] free.

⚠ Warnings

- A separate exhalation device must be used with this mask.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. **Do not block or try to seal the exhalation port.**
- **Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. **Explanation of the Warning:** When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, nasal mask selection, and the leak rate. This warning applies to most types of CPAP machines.
- Some users may experience skin redness, irritation and/or discomfort. If this happens, discontinue use and contact your health care professional.
- This mask is not suitable for providing life support ventilation.

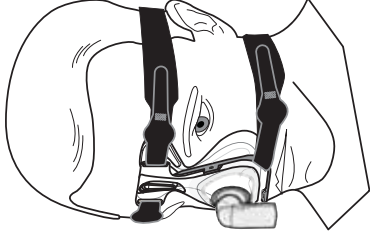Symbols

Caution or Warning

Consult instructions for use

Before Use

- Read and understand the instructions completely.
- Hand wash the mask.
- Inspect the mask components for damage or wear; replace if necessary.

Cleaning Instructions for the Mask and Headgear

For **multi-patient, multi-use** in the hospital/institutional environment, visit us on-line at www.sleepapnea.com or contact Customer Service at 1-724-387-4000 or Respironics Deutschland at +49 8152 93060 to obtain mask disinfecting instructions.

For **single patient reuse** in the home, the mask and CPAP tubing should be cleaned daily. The headgear should be cleaned weekly, or as needed. The headgear does not need to be removed for daily cleaning. Inspect the headgear for wear and tear, replace as needed.

- Hand wash in warm water with a mild liquid dish washing detergent. Do not use bleach, alcohol, or cleaning solutions containing alcohol. Do not use cleaners containing conditioners or moisturizers.
- Rinse thoroughly. Air dry completely before use.
- Inspect all parts for damage or wear; replace if necessary.

EasyLife Mask

The EasyLife Mask is designed with two mask cushions that work together.

- The inner cushion provides a comfortable and automatically adjusting seal.
- The outer cushion provides stability and support to the mask. The outer cushion is not designed to provide seal.

Achieving the Right Fit

1. Have the patient hold the mask with the inner cushion over their nose and pull the headgear over their head. The headgear is preset to a large setting to go on easily right out of the package.
2. Have the patient evenly adjust the upper straps (Figure 3) to a comfortable position by using the tabs on both sides of the headgear. Do not pull the straps tight. The cushion should lightly touch the face and the upper strap will rest above the ears.
3. Evenly adjust the lower straps (Figure 4) to a comfortable position by using the tabs on both sides of the headgear. Do not pull the straps tight. The lower strap will sit below the ears and the headgear will cross at the base of the head (Figure 5).
4. Attach the separate exhalation device (e.g.: Whisper Swivel) to the blue standard elbow (SE).
5. Connect the tubing (included with the CPAP or bi-level device) to the standard elbow.
6. Turn on the airflow.
7. Have the patient lie down and breathe normally through the nose, keeping lips closed.
8. Adjust the headgear evenly to reduce leaks or pressure points at the bridge of the nose or above the lip.
9. Reseat the mask, if necessary.

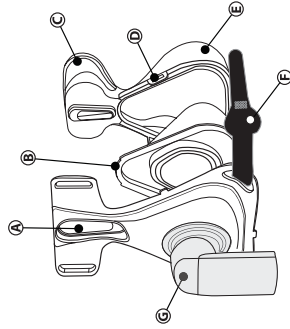
Figure 2

A = Forehead pad slot
B = Inner sealing cushion with blue flap
C = Forehead pad

D = Attachment tabs
E = Outer support cushion
F = Headgear stop
G = Blue SE Standard Elbow
no built-in exhalation

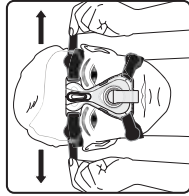
Figure 3

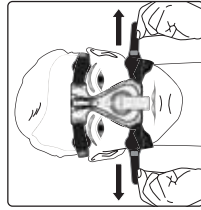
Figure 4

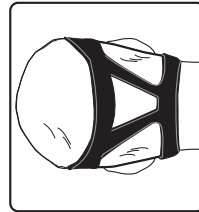
Figure 5

Comfort Tips

- The headgear should fit loose and comfortable. If your skin bulges around the mask, loosen the headgear. Over tightening the headgear can hinder the mask's performance and seal.
- Reseat the mask by pulling the sides of the faceplate directly away from the face then gently set it back into place. This allows the cushion to create a new seal on the face. **DO NOT pull the mask by the elbow.**
- Adjust the upper headgear straps to eliminate leaks around the forehead and temples.
- Adjust the lower headgear straps to eliminate leaks around the cheeks and chin.

Removing the Mask

Quickly and easily remove the mask by grabbing the back headgear strap and sliding the entire assembly forward over your head. If needed, undo the lower headgear straps. The headgear stop prevents the headgear from slipping out of the frame. You do not need to pull the headgear straps beyond the stops for daily cleaning or adjustments.

Removing and Attaching the Cushions

To Remove

1. Pull outward on the three attachment tabs on each side of the outer support cushion until this cushion separates from the mask frame.
2. Remove the outer support cushion.
3. Remove the inner sealing cushion with the blue flap.

To Attach

1. Place the smaller inner sealing cushion on the mask frame (Figure 6).

Note

The blue flap must cover the lip on the mask.

2. Slide the larger outer support cushion with tabs facing downward around the inner sealing cushion until all three sides click into the mask frame (Figure 7).
3. Gently tug on the cushions to make sure they are securely in place.
4. Gently press the forehead pad into the forehead pad slot in the mask frame.

Attaching the Headgear to the Mask

1. To reattach the headgear, thread the upper headgear straps into the slots in the top of the mask frame. Fold the tabs back to catch the headgear straps.
2. Thread the lower headgear straps into the slots in the bottom of the mask frame. Fold the tabs back to catch the headgear straps.

Note

The upper headgear straps are longer and the lower headgear straps are shorter (Figure 8).

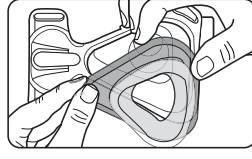
Figure 6

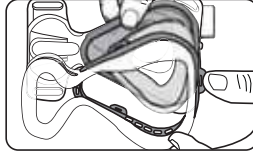
Figure 7

Figure 8

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